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**PLEASE READ WHOLE INSTRUCTION
CAREFULLY BEFORE USE.**

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Introduction

Thank you for choosing our product. We are deeply convinced that it will meet your expectations. Please read this document before first use. This manual is passed in order to provide the necessary information for the safe use of the device. The correct application of the recommendations will contribute significantly to extend the durability and aesthetics of the product. We want to emphasise that constantly we are trying to improve the quality of our products, which may result in minor changes that are not included in the manual

Product description

Anti-bedsore cushion consist of numerous, elastic, air interconnected cells provide proper pressure distribution which prevents skin abrasions and bedsores. Inflating and pressure control is possible due to hand pump and air valve.

Cover made of breathable, perforated fabric and empty spaces between the cells provide optimal comfort of use and air flow - warmth is taken away from the body which prevents excessive perspiration. The cushion is easy to maintain, highly hygienic.

Additionally, the cells in two-zones version are divided into two zones. Each of the zone has separated valve. Thanks to such construction its possible (depending on cushion position-transverse or longitudinal) to correct patients position or adjust tilt angle.

Anti-slip pad maintains the cushion on place.

Application

The cushion prevents skin abrasions caused by sitting for long periods, relieves lumbar spine, provides proper pelvic position which prevents pain related to lumbar spine overloading.

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Contrainications

Contraindications to a sitting position, bedsores V stage

Use

- Take the cushion out of the box and twist the valve counter-clock wise to open.
- Attach silicon hose of pump to valve. Inflate the cushion with pump.
- Close the valve by turning it clockwise.
- In order to regulate softness and height of the cushion, sit on the cushion and turn the valve clockwise.
- Adjust the cushion softness to individual needs to obtain maximum comfort.
- Readjusting the cushion pressure is recommended every 10 days.
- In case of two-zones cushions, repeat actions 1-4 for the second valve.

Safety of use

WARNING !

- а) Before using the cushion, refer to this manual and keep it for future reference.
- б) Permitted user weight 130 kg.
- в) Avoid using the cushion in extremely high or low temperature, do not expose it to UV rays.
- г) Keep away from fire.
- д) Keep away from sharp objects.
- е) The cushion can be used only with original cushion cover.
- ж) Please remember that on the cushion should stay only the patient (other people on the mattress are not allowed).
- з) Pay attention to skin changes. If necessary, consult your doctor or nurse.
- и) The cushion can be used on the flat surfaces only. Using the mattress on the orthopaedic adjustable beds it is not recommended.

Everyday use and maintenance

For hygiene reason anti-bedsores cushion should be used only by one patient.

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The cushion cover should be washed up by hand in warm water using mild detergents. Do not chlorinate. Do not whirlpool, wring gently before drying. Dry in horizontal position. Dry in a distance of artificial sources of heat. Do not iron. Close the valve before cleaning the cushion. The cushion should be washed up with the damp cloth using mild detergents. Dry in a distance of artificial sources of heat. Do not iron. Always put on the cover when the cover and cushion are completely dry.

Technical parameters

Cushion available in the following sizes:

- 40 cm x 40 cm x 8 cm two-zones version
- 46 cm x 40 cm x 8 cm two-zones version
- 46 cm x 46 cm x 8 cm two-zones version
- 50 cm x 46 cm x 8 cm two-zones version

130 kg

COMPOSITION:

Cushion : PCV

Cushion cover: 100% poliester

Repair

- In case of reparation, please contact the producer.

Customer service

- If you have any questions or need help, contact the producer.

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